DE Helimatic® Disinfectant

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GB Helimatic® Disinfectant

Do not breathe mist. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Avoid release to the environment. Dispose of contents/container to an approved waste disposal plant.

Information for use: Reusable medical devices should be reprocessed in line with the manufacturer instructions. Use Helimatic® Disinfectant on pre-cleansed medical devices only.

When Helimatic® Disinfectant concentrate used in a washer/disinfector (WD), a disinfection effect achieved at an application concentration of at least 10 ml / L at 55 – 60°C and an exposure time of at least 5 minutes. The program selected acc. to the manufacturer's instructions. Dosing takes place via the machine's own dosing pumps.

Caution: Do not use in combination with alkylamine based disinfectants!

Storage and handling conditions: Do not store above 25 °C.

Notice to the user: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is established. For professional use.
Date of issue of the instruction for use: 2021-01-11
Date of the last revision: 2021-01-11

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PZN 03481098

UN 1903

Composition: 100 g solution contains 20 g glutaral, solvents, salts of organic acids, corrosion inhibitors, excipients
Contains: **Glutaraldehyde**
UFI:1E7V-U711-0009-TTES

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GB Danger. Corrosive to the respiratory tract. Harmful if swallowed or if inhaled. Causes severe skin burns and eye damage. May cause an allergic skin reaction. May cause allergy or asthma symptoms or breathing difficulties if inhaled. May cause respiratory irritation. Harmful to aquatic life with long lasting effects.

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34212 Melsungen, Germany

Tele +49 (0) 5661 710

B. Braun Medical AG, Seesatz 17, 6204 Sempach, Switzerland

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B. Braun Melsungen AG