




2-piece system

FLEXIMA® 3S

LATEST GENERATION OF MECHANICAL COUPLING SYSTEM

Flexima® 3S

As simple as it gets

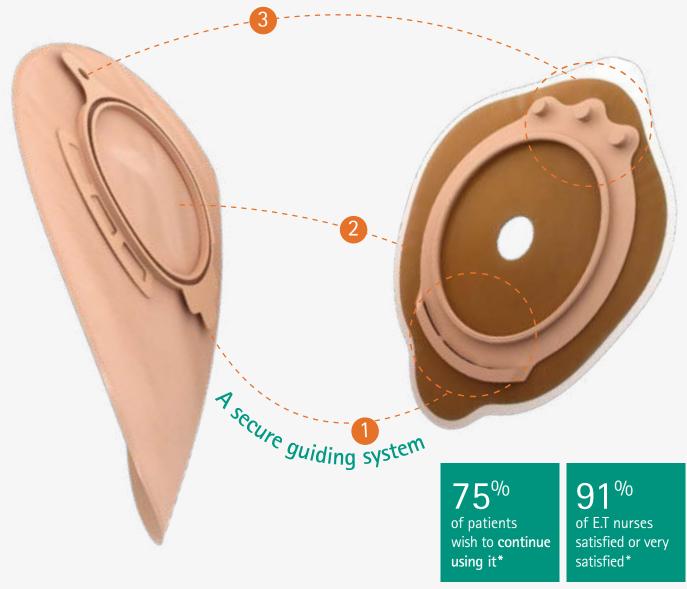


A complete range to fulfill each patient's needs**



^{**}Availability may vary depending on the country.

Flexima® 3S A secure range


High performance filter

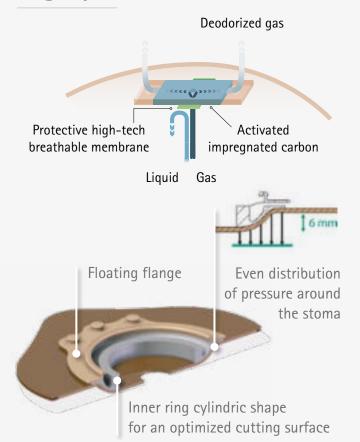

 Leak-Proof and permeable to gases high-tech breathable membrane

Flexima® 3S wafer

- Largest diameter available in flat: 80mm
- 6mm convexity with a large base

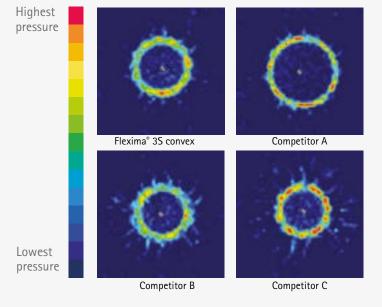

Pressure Mapping* of 2-piece convex wafers


Flexima® 3S convex wafer has been designed to provide a sufficient level of pressure, evenly spread out on the peristomal area.

*Source: R&D data on file Test Report CR0663 (May 2014).


Flexima® 3S Closed and drainable Roll'Up bags

Compact shape for more discretion

Closed bags

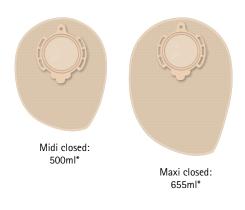

Drainable Roll'Up bags

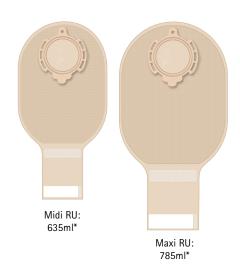

Integrated soft Roll'Up clamp

Intuitive and secure closing system



Soft edges



• A large 7cm outlet for • White opening tabs both colostomates and ileostomates



Flexima® 3S High flow bag




High capacity bag

- Compact shape
- Large funnel outlet for direct flow: Ø13.3 mm
- Large diameter of the outlet for a direct connection to the Flow Collector bag

Both High-Flow outlet and Flow Collector bag have been specifically designed together to provide a safe answer to high-output indications and an easy to handle system.



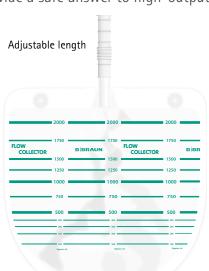

Flow Collector

- Compact shape with a 2 liter capacity
- White back cover
- White opaque tube (1,5m)
- Printed approximative scale for a volume measurement
- Hooks available for hanging or attaching the bag

- Outlet located at the bottom of the bag
- Large diameter for an emptying of output including bits (20mm)
- 2 notched tabs available: one at the top and one on the side

Flexima® 3S Uro bags

- Ergonomic & compact shape
- Anti-reflux valve
- Specific B. Braun tap



92% of uro patients rated Flexima® 3S Uro "good" or "very good" on absence of leakage.

*Field Acceptance Test on panel of uro patients, France from September 2012 to January 2013.

Available in beige & transparent

The system is extremely safe. It's easy to fit, no leakage between the wafer and the bag. The possibility to use a belt is very handy.

Enterostomal Therapist

No need for visual check to know if outlet is closed or open.

A Secure outlet

OUTLET FOR DAY

- O Position open/closed
- Fits in perfectly strengthened by a click and a final locking system
- Easy handle; no risk of wet fingers

OUTLET FOR NIGHT

- Click at connection
- Can be connected to a night drainage bag
- Outlet designed to avoid bag twist



Simple: it guides into position for you

Right positioning from the start

- 1. Insert the tab into the wafer slot
- 2. Clip the bottom of the ring and slide it up
- 3. Lock the pinpoint to secure the system

Minimum patient education required Multiple orientation

The guiding system will completely revolutionize the patient's life: the insertion of the guiding tab into the slot is very intuitive. No need to think twice!

Enterostomal Therapist



89% very satisfied or satisfied with guiding system*

Secure: it gives you confidence!

Enhanced leak proofing


• Thanks to the close fitting mechanical coupling system

Increased security

 Strenghtened by both the click and the final locking system

Optimal belt fixation

Three hooks on the bag add stability when needed

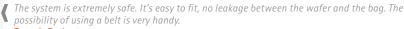

French Patient



84⁰/₀ Secure feeling during wear*

Soft: it becomes a part of you!

Minimum pressure on the abdomen

• The floating flange prevents discomfort during the application

High degree of comfort

• Flexible when wafer and bag are coupled together

Optimised body fit

 New generation of good plastic material allowing for great discretion

{{ It is simply perfect. It is flexible, as much as a 1-Piece system can be. **Enterostomal Therapist**



91 % wearing comfort (flexibility and discretion)*

Flexima® 3S

FLEXIMA® 3S FLAT - BOX OF 5

	ø (mm)	References
*	45/ 15-30	936415
	45/ 20	936420
	45/ 25	936425
	45/ 30	936430
*	55/ 15-40	936515
	55/ 35	936535
	55/ 40	936540
*	65/ 15-50	936615
	65/ 45	936645
*	80/ 15-65	936815

FLEXIMA® 3S CONVEX - BOX OF 5

%	45/ 15-26	937415
9 3	45/ 25	937425
*	55/ 15-36	937515
	55/ 30	937530
*	65/ 15-46	937615

FLEXIMA® 3S HIGH FLOW - BOX OF 30

Size	ø (mm)	References	Volume
Maxi	55		. 0.70 m.l
		O 933155	±970ml
	65		. 0.40
		O 933165	±940ml
	80	933080	. 000
		○ 933180	±900ml

FLOW COLLECTOR®

039900	BOX OF 15 ±21
039901	BOX OF 5 ±21

FLEXIMA® 3S CLOSED - BOX OF 30

Size	ø (mm)	References	Volume
Midi	45	931045	±520ml
	55	931055	±500ml
	65	931065	±465ml
Maxi	45	931445	+665ml
		O 931545	±0001111
	55	931455	CEEml
		O 931555	±655ml
	65	931465	±630ml
		○ 931565	
	80	931480	. E10ml
		○ 931580	±510ml

FLEXIMA® 3S ROLL'UP - BOX OF 30

Midi	45	932045	. CEOmal
		O 932145	±650ml
	55	932055	+635ml
		O 932155	±0351111
	65	932065	+615ml
		O 932165	±0131111
Maxi	45	932445	+805ml
		O 932545	±8031111
	55	932455	. 70Eml
		O 932555	±785ml
	65	932465	±755ml
		O 932565	
	80	932480	. 71 Fml
		○ 932580	±715ml

FLEXIMA® 3S URO - BOX OF 30*

Midi	45	934045	+675ml
		O 934145	±0/3III
	55	934055	+635ml
		O 934155	±0351111
	65	934065	+620ml
		O 934165	<u> </u>

^{*}with 30 universal connectors

 \mathscr{C} Cut-to-fit $| \bullet$ Beige $| \bigcirc$ Transparent $| \bullet$ Beige with inspection window For most countries a specific letter is added at the end of the radical reference

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.

Commercialized and manufactured by :

B. Braun Medical SAS | 26 rue Armengaud | 92210 Saint-Cloud - France
Tél. 01 41 10 53 00 | Fax 01 70 83 45 00 | www.bbraun.fr | www.bbraun.com
Societé par Actions Simplifiée with a share capital of 31 000 000 € | RCS Nanterre 562050856



All stoma bags volumes are measured according to ISO 8670-2 standard.

This medical device is a regulated health care product and on the basis of this regulation is certified CE.

The products commercialized by B. BRAUN MEDICAL shall exclusively be used in accordance with the package inserts which have been updated and included in the boxes. Non binding documents and photographs