# Actreen® Mini Set

**EN** Instructions for use  
**DE** Gebrauchsanweisung  
**ES** Instrucciones de uso  
**FR** Mode d'emploi  
**IT** Istruzioni per l'uso  
**NL** Gebruiksaanwijzing

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**B | BRAUN**

# EN Instructions for use

Actreem® Mini Set is a sterile, pre-lubricated, ready-to-use set, specifically designed for female intermittent catheterisation.

Unfold the set by removing the white sticker on the back. **A**

## Preparation:

- 1 - Carefully wash hands before and after catheterisation, and clean the area surrounding the urethral opening.
- 2 - Before opening the catheter, check that the pouch is intact and inspect it (visually and by touch), also check the catheter to ensure the lubricant is spread evenly.
- 3 - Open the catheter by tearing and removing the pre-cut protective sheath. **B**
- 4 - Hold the catheter by the connector without touching the catheter.

## Catheterisation:

- 5 - With one hand, separate the labia and with the other, gently insert the catheter through the urethral opening into the urethra **C** and up into the bladder, until urine starts to flow into the bag.
- 6 - Manual pressure on the lower abdomen towards the end of catheterisation will ensure that the bladder is completely emptied.
- 7 - When the bladder is completely empty, gently withdraw the catheter.

## Emptying:

- 8 - To empty the bag, tear off along the dotted line located on the top. **D**  
To discreetly dispose of Actreem® Mini Set, refold the set and reposition the sticker to close it. **E**  
Then discard the catheter according to the current local regulations.

Self-catheterisation should only be carried out under medical advice and only in accordance with the instructions given.

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B. Braun Medical AB
B. Braun Medical SAS | 204 av. du Maréchal Juin | 92107 Boulogne Cedex | France