AESCULAP® AdTec® MINI MONOPOLAR DISASSEMBLY | ASSEMBLY INSTRUCTIONS






DISASSEMBLY




I Deactivate ratchet on ratchet handle Flip down red toggle switch. N.B.: for PO381R only

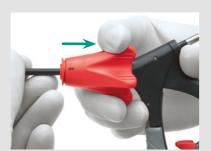


Disconnect shaft and handle Pull rotation knob back and let the movable part of the handle move downwards.




I Remove shaft from handle Remove the shaft by pulling the instrument shaft out of the handle.

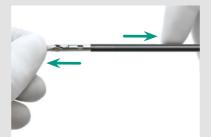

Disassemble shaft I Hold the jaw insert at the tip and remove the outer insulation tube.

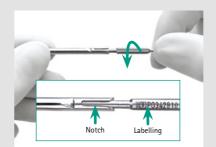


I Disassemble shaft II Rotate the push rod 90° counterclockwise until the labelling is in line with the notch of the jaw part.

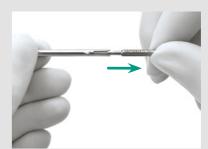


I Disassemble shaft III Remove the push rod.


ASSEMBLY

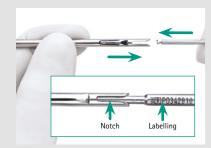

Assemble shaft I Slide the push rod into the jaw insert as far as it will go making sure that the labelling on the push rod is in line with the notch of the jaw part.

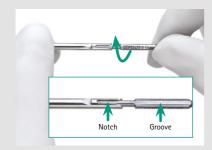


Assemble shaft II Rotate the push rod 90° clockwise until it locks into place. The groove is then in line with the notch.

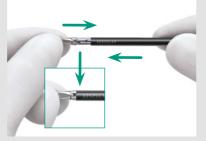

Assemble shaft III Slide the black outer tube as far as it will go over the jaw insert.



Prepare handle Deacitvate the ratchet by flipping down the red toggle switch. N.B.: for PO381R only

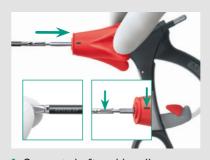

I Connect shaft and handle Hold the tip of the shaft and the rotation knob of the handle firmly. Slide the shaft into the handle until it locks into place, making sure that the push rod notch is in line with the mark on the rotating knob.



I Function test Test the assembly of the instrument by opening and closing the handle to ensure the jaw is openinig and closing.

Important: Do not touch the movable part of the handle

CARE






Lubricate the defined places on the instruments (instruments jaw and red toggle switch) with AESCULAP® STERILIT® Oil I For accessories and spare parts, please refer to after each cleaning cycle.

PROCESSING & ACCESSORIES

- I Reprocess the product immediately after use
- I The product can be sterilized either in disassembled or in assembled condition
- For advice on cleaning, disinfection and sterilization, please follow the instructions for use
- For the latest information on reprocessing, please also refer to the Aesculap Extranet via www.aesculap-extra.net
- brochure C28011 and C28511 or visit www.endoscopy-catalog.com

REPAIRS

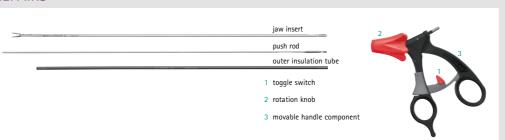

In order to ensure the preservation of full guarantee and warranty rights, only persons authorized by Aesculap may carry our renairs.

Please send products for repair to: Aesculap Technical Service, Am Aesculap-Platz, 78532 Tuttlingen, Germany. Additional service addresses may be obtained from the same address or via ats@aesculap.de

C28102 0721/PDF/3