
Effective

Flow Collector




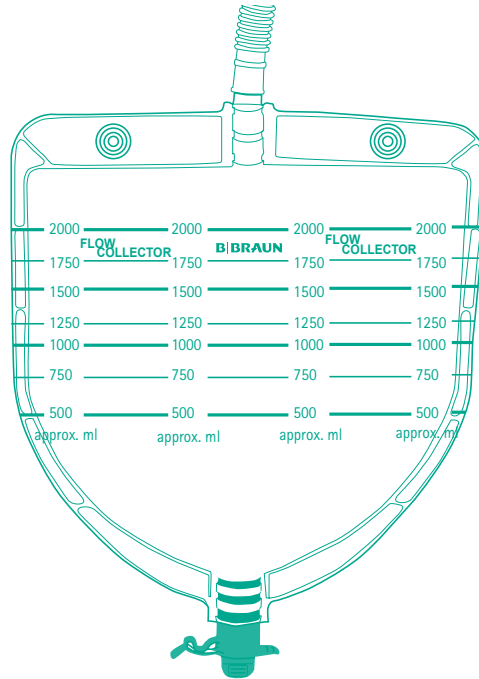
Effective

B | BRAUN

Effective

Flow Collector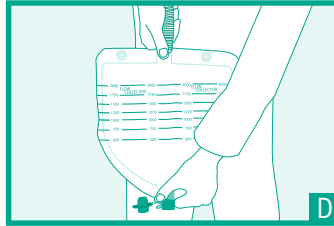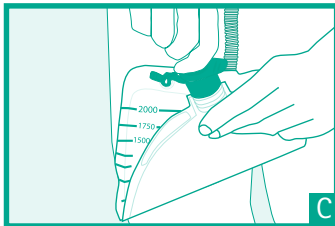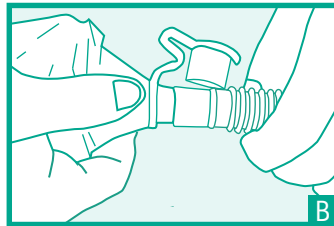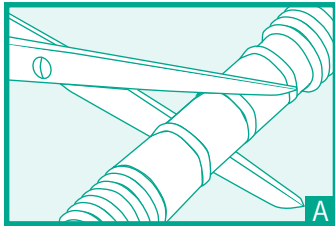

Effective

EN Instructions for Use**What is Flow Collector and what it is used for:**

Flow Collector is a drainable collection system for high flow liquid stools. This product ensures the collection of high flow liquid stools being connected to the stoma bag of the patient.

It is to be used with the Almarys® Twin+ pouches for High Flow, Flexima®/Softima® pouches for High Flow, Flexima®/Softima® Key Maxi and Flexima®/Softima® 3S for High Flow.

The collector can be placed on the floor or higher than the floor. It must always be lower than the stoma bag.

Flow Collector may be used concomitantly with B. Braun stoma care accessories.

Intended purpose and indication:

The product is indicated for collection of high flow liquid stools being connected to the stoma bag of the patient.

Flow Collector is intended for ileostomates, jejunostomates and colostomates with liquid stools.

Clinical benefit & performances:

Use of Flow Collector reduces the frequency of appliance drainage procedures.

Patient population: enterostomates adults and children above four years. Enterostomates with high volumes of stoma output requiring a collection system with large storage capacity.

Contraindication:

There are no formal contraindications for the use of Flow Collector.

Wash your hands before starting the procedure.

1. Fitting the appliance

- Adapt the length of the tubing to the desired size with scissors by cutting it in the middle – against the edge – of the smooth areas (cutting is possible every 30 cm) (A).
- Close the green drainage cap by pressing it before the connection.
- Connect the tubing to the drainage system of the stoma bag by placing the outlet of the pouch inside the Flow Collector's tube up to the stop (B).

2. Disconnecting after use

- Pull on the tubing while pressing it slightly and close the drainage system of the stoma pouch.
- One Flow Collector can be used for a duration of about 72 hours.

N.B.:

- It is possible to get an approximate measurement of the effluent volume thanks to the scales in the front of the collecting pouch. For a more precise evaluation, use a measuring collector.
- It is possible to use B. Braun Ileo Gel+ by introducing 2 to 4 sachets in the Flow Collector (through the green cap at the lower end of the pouch). It solidifies stools: the pouch is then more silent, and the emptying is easier.

3. Using the drainage system

For an easier emptying, lay it on the toilet seat (C).
Open the cap using the grip and taking good care to maintain it higher than the pouch.

Maintain the cap between the thumb and the index finger during the emptying (D).

Once the collecting pouch is empty, clean the cap with tap water or with a wipe (caution: do not use fluffy material) to maintain leakproofness.

National regulations for the destruction of potentially infectious materials must be observed.

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B. Braun Medical Oy
CO.,LTD
B. Braun Medical AB
B. BRAUN MEDICAL SAS | 26 Rue Armengaud | 92210 Saint-Cloud – FRANCE